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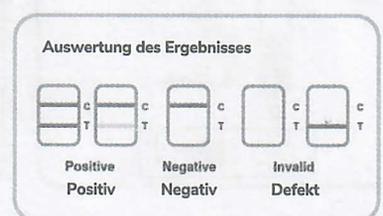
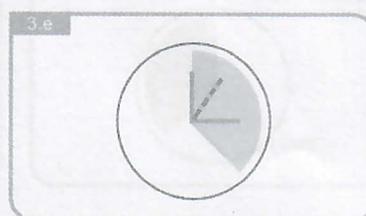
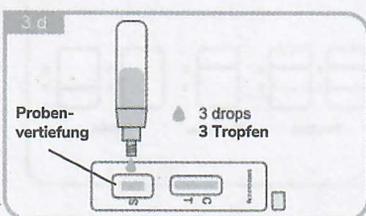
TEST PROCEDURE

The SARS-CoV-2 antigen is generally detectable in upper respiratory samples during the acute phase of infection. The viral load in oral fluid specimens is significantly lower than nasal (NS) swab specimens, please follow the instruction below if processing with oral fluid specimens.

1. Precautions:

- Oral fluid specimen must be freshly collected.
- Do collect samples as soon as possible after the onset of symptoms.
- Do test collected samples immediately.
- The sample is best collected after waking up in the morning.
- Do not eat or drink within 1 hour before sampling.
- Optional step 2.d and 2.f can increase the accuracy of the test.

2. Specimen collection, handling, and testing:

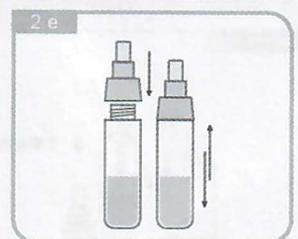
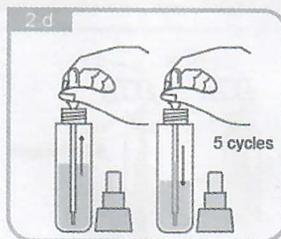
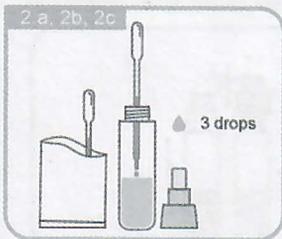
1) Specimen collection

- a) Before collecting oral fluid, relax your cheeks and gently massage cheeks with fingers for 15-30 seconds. Place your tongue against the upper jaw to enrich the oral fluid.
- b) Open the oral fluid specimen collection bag, spit into the collection bag as shown in the following figure.

2) Specimen handling

- a) Remove the cap of extraction tube.
- b) Hold the dropper vertically and aspirate oral fluid from collection bag.
- c) Transfer three (3) drops of the collected oral fluid into the extraction tube. *Discard the remaining liquid in the dropper back to the collection bag. Make sure the dropper is empty before proceeding to step 2.d.
- d) Use the dropper to mix the liquid in the extraction tube by aspirating and dispensing for at least 5 cycles.
- e) Cap the lid of extraction tube. Swirl or flick the bottom of the tube for at least 15 seconds to make sure the oral fluid mix with the buffer thoroughly.
- f) The treated sample solution should be used within 5 minutes and should not be stored.

3) Specimen testing

- a) Tear off the foil pouch, take out the test cassette and place the test kit on a clean and level surface.
- b) (Optional) Label the test device and one extraction tube for each specimen or control to be tested.
- c) Remove the nozzle cap from extraction tube.
- d) Gently squeeze the ridged body of the tube, dispense **three (3) drops** of the processed specimen into the sample well.
- e) Read the test results between **15 and 20 minutes**. Do not read the results **after 20 minutes**.